


4 BRP
Bio Rivitalization Points



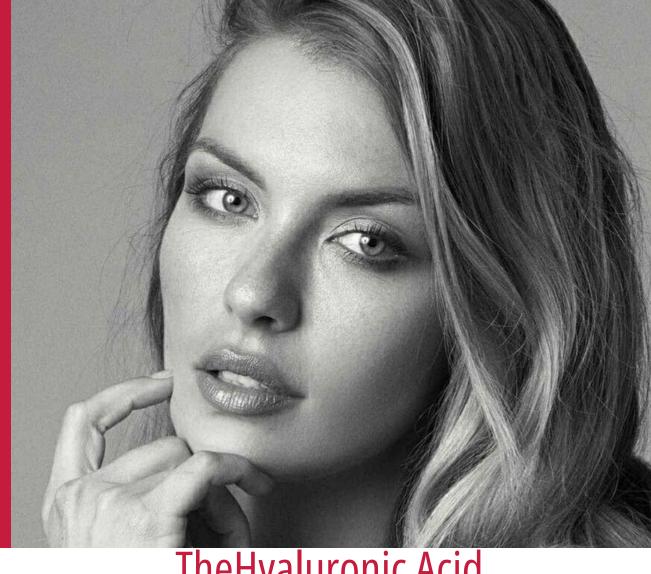

The Hyaluronic Acid





The box of Ejal 40








The innovative formula



The innovative formula of EJAL40 provides for the addition of a quantity and of different molecular weights of Hyaluronic Acid in 3 steps to each work process. This allows for better elasticity and longer life.

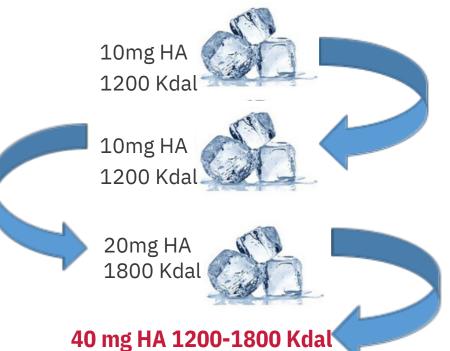





4 Reasons to choose Ejal 40




EJAL 40 is the latest generation of hyaluronic acid with medium molecular weight produced to rebuild the basic structure of the extra cellular matrix (ECM) so that the cells involved in the regeneration phase of new bers can establish again the natural functional relationships.

EJAL 40 thanks to its effectiveness is able to restructure and reinforce the extra cellular matrix revitalizing the skin already dehydrated and damaged by aging and solar exposure.

EJAL 40 increases the biosynthetic capacity of fibroblasts inducing the regeneration of an optimal physiological environment, enhancing the cellular activities, hydration, and synthesis of collagen, elastin and hyaluronic acid.

EJAL 40 is an innovative hyaluronic acid formulated with a concentration of 40 mg / 2 ml in a physiological, sterile, biodegradable and isotonic buffer. A syringe pre- lled contains 2.0 ml of gel, sterilized.

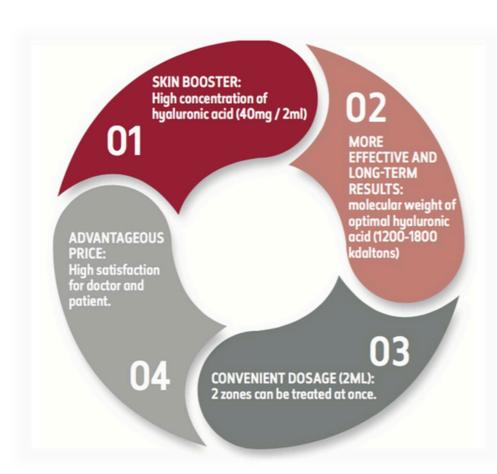




The repairing process of the aged skin



EJAL 40 bio-revitalizing gel is another step ahead in this direction: it restores the physiological functions of the skin and improves skin compactness by restructuring the extracellular matrix thanks to the stimulation of the activity of fibroblasts and to the creation of new collagen and elastin bers.

The repairing process of the aged skin is a very complex biochemical dynamic event, and it has been studied over the last few decades during which many success has been achieved thanks to the effectiveness of hyaluronic acid, a fundamental element of the extracellular matrix, which generates an optimal micro-environment able to stimulate the proliferation and migration of fibroblasts, endothelial cells, keratinocytes, and neoan- giogenesis.

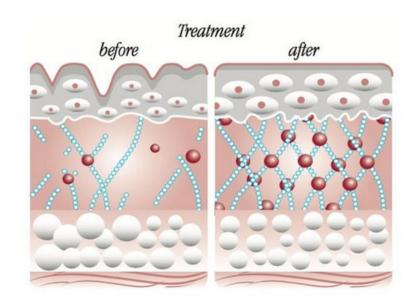




Natural decrease of hyaluronic acid (HA)



Skin appearance is a primary indicator of aging, one of the main characteristics is the fragmentation of the extra cellular matrix (ECM) and collagen bers. In fact, collagen fragmentation is responsible for loss of structural integrity and impairment of broblast functions in the skin. Treatments with hyaluronic acid that stimulate the production of new collagen will provide a substantial improvement to the appearance and health of aged skin. EJAL 40 reintegrates the level of hyaluronic acid (HA) by counteracting its natural decrease due to the aging process and it is perfectly indicated in tre- atments of skin rejuvenation, prevention of photo-a- ging damages, stretch marks treatment and before the use of intradermal llers.

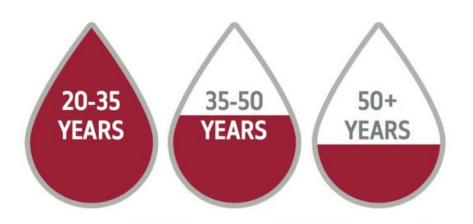

Natural decrease of hyaluronic acid (HA)



4 points for biorevitalization **₹** BRP

The most common treated areas are submural and malarial. In particular to treat these areas, we suggest to use EJAL 40 with a more innovative 4 point biorevitalization technique. 4 BRP - 4 Bio Revitalization Points technique allows you to use a 0.25 ml injection with a 30G needle on each side of the face with only 4 points, minimizing the risk of blood leakage and optimizing the effectiveness through the most homogeneous and fluid injection of hyaluronic acid. It is suggested to apply the technique following the protocol.



Protocol



- 3 Treatments at a distance of one month followed by a two-month maintenance depending on the aging level.
- Repeat the cycle twice per year. 1 month 1 month EJAL 40 is particularly suitable for the treatment of skin laxity, wrinkles, decreased tone and elasticity and before the use of intradermal llers.





Ejal 40 Exclusion criteria



- Tendency to the development of
- hypertrophics scars
- Autoimmune diseases or immune therapy
- Hypersensitivity to product components
- Pregnancy or breast-feedingAge Under 18 years.

Treat patients on anticoagulants with EJAL40 or with anti-medication only after consulting your doctor Curante
Do not use EJAL40 in areas with skin processes inflammatory and / or infectious (eg acne, herpes, etc.).





Ejal Cosmetic

























































